



Invacare® Twilight™ Mask Cushion Size Guide

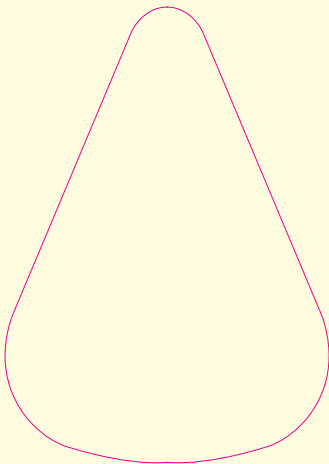

Invacare Twilight Mask order information:

ISP2000	Twilight Mask	Standard size
ISP2000L	Twilight Mask	Large size
ISP2010	Replacement Twilight Mask Cushion	Standard size
ISP2010L	Replacement Twilight Mask Cushion	Large size

Invacare® Twilight™ Mask Cushion Size Guide

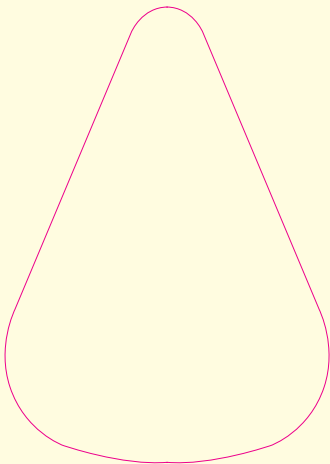

Directions:

Place card over patient's nose. Proper fitting for standard size cushion is identified when the top of the triangle contacts the nose at or above the valley between the nose and the forehead. If the patient's nose exceeds this triangle, then the large size cushion is required.



Yes, you can.™